



EYEDAPTIC TECH NOTE: AUTOFOCUS

The Eyedaptic glasses have autofocus capabilities, meaning the camera will automatically adjust the focus according to the camera's subject and the subject's distance from the lens.

While the autofocus does work successfully most of the time, occasionally the camera will get “stuck” out of focus. This will result in slight blurriness seen through the displays. However, there are two recommended methods to force the camera to re-focus and bring a clear image back. These are explained below:

1. Switch between the glasses camera and the phone camera by activating EyeSwitch mode
 - a. On the homepage of the Eyedaptic app, click on the  button on the top left of the screen
 - b. This will switch your view from the camera in the center of the glasses to the camera on the back of the phone
 - c. Then, click the  button again to switch back – this should force the glasses camera to re-focus

2. Bring your hand up close, right in front of the glasses camera, and then quickly pull away – this should force the glasses camera to re-focus

