



EYEDAPTIC TECH NOTE: EYESWITCH®

EyeSwitch® is one of the most unique features of Eyedaptic® and can be one of the most helpful!

EyeSwitch® allows you to utilize both the glasses camera and the phone camera. The phone camera has a higher resolution than the glasses camera, making it ideal for reading small text or details. By activating EyeSwitch® mode, the phone essentially becomes a handheld magnifier or CCTV!

EyeSwitch® mode can be activated either through the phone or through the glasses.

How to activate EyeSwitch® mode (using the phone):


1. Click on the  button in the top left of the Eyedaptic® app homepage
2. Now, lift the phone up and point the camera on the back of the phone towards what you want to read or look at
3. To switch back to the camera on the glasses, simply click on the  button at the top center of the screen

How to activate EyeSwitch® mode (using the glasses):


1. There is a single button on the top of the glasses – press and hold this button for approximately 1 second, and then let go
 - a. This will activate EyeSwitch® mode, so now you will lift the phone up and point the camera on the back of the phone towards what you want to read or look at
2. Repeat the same process to switch back to the camera on the glasses - press and hold the button on the glasses for approximately 1 second, and then let go



*Note: when in EyeSwitch® mode, you will not have access to the entire homepage, but you can still magnify with the zoom slider and activate both Autozoom and Britext.

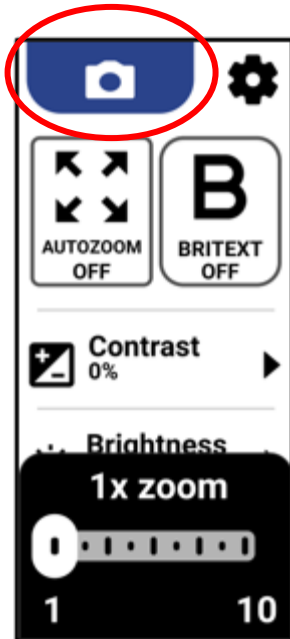

EyeSwitch® mode NOT activated

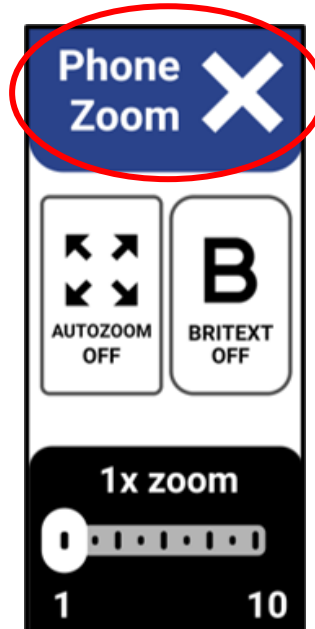

EyeSwitch® mode activated

When to use EyeSwitch® mode:

1. When reading fine print – ie. Medication bottles, ingredients on the back of a can, etc.
2. Watching TV – we recommend purchasing a [phone stand](#) on Amazon for this purpose
 - a. You can place the phone in the stand, point it towards the TV, and relax
3. Fine detail work – ie. Sewing, painting, etc.