


## EYEDAPTIC TECH NOTE: PAIRING THE HANDHELD REMOTE CONTROL

### How to pair the handheld remote control using Bluetooth:

\*This process only needs to be done once per remote control and phone – after it is done the first time, the phone will remember the pairing every time you turn the remote control on

1. With the phone turned on, go to the **phone** settings app
  - a. If you are on the phone home page, you will see the Eyedaptic icon – from here, click the settings button at the bottom left of the screen
  - b. If you are in the Eyedaptic app, you can exit to the phone home page by swiping up from the bottom and clicking the circle icon and then continuing to (a)

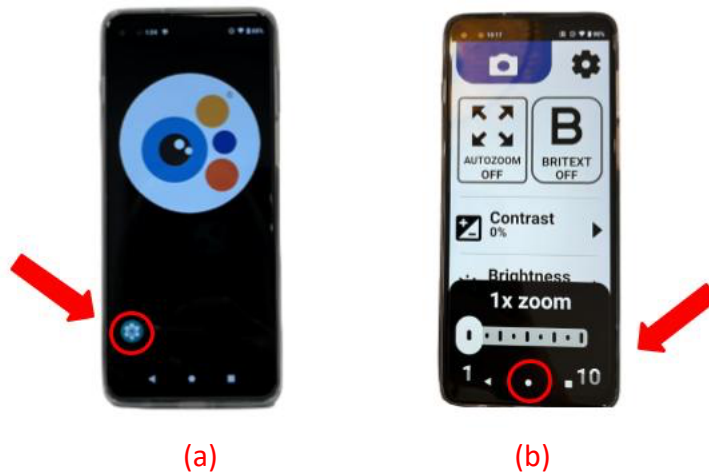

2. Once in the **phone** settings app, select “Connected devices” and select “Pair new device”
3. Turn on the remote control by holding down the button in the center, until a green light comes on



4. After turning the remote control on, you should see a device pop up under “Pair new device” that is called BTXXXX (ie. BT009)
5. Select this device, click “Pair” when a notification pops up, and wait until you see that the connection is made (underneath the BTXXXX, you will see the battery percentage of the fob)
6. You may now use it with the glasses (instructions for navigating the remote control are below)

How to charge the handheld remote control:

1. In the box that you receive your remote control in, there is a charging cable underneath the white plastic piece



2. To charge, plug one end of this cable into the charging block that comes with the Eyedaptic companion phone, plug the other end into the remote control charging port, then plug the block into any outlet
  - a. The charging port for the remote control is located on the back side of the device, and can be found by removing the remote control from its plastic holder



## Remote Control Overview

\*Optional additional accessory

★ Hold to turn fob on/off

★ Toggles between manual zoom and autozoom

Manual zoom increase



Cycle through contrast modes

- ★ Black on white
- ★ White on black
- ★ Color (normal)

Toggles Britext on/off

Manual zoom decrease



Green light: device is on



Red light: fob needs to be charged

1